

Troubleshooting Error Codes

1. Film Session errors

check the following :

- ▶ Number of copies
- ▶ Print Priority (NAI default is medium)
- ▶ Media type (Film, Paper)
- ▶ Destination (Processor, Magazine)

2. Film Box errors

check the following:

- ▶ Format (number of images per sheet e.g. 3x4=12)
- ▶ Orientation (Portrait, Landscape)
- ▶ Media size {8x10 in: 10x12 in: 10x14 in: 11x14in: 11x17 in: 12x18 in:
(14x14 in, 35x35 cm): (14x17 in, 35x43 cm): 24x24 cm: 24x30 cm:}
- ▶ Magnification type / Scaling (cubic, none, replicate, bilinear)
- ▶ Trim (off, on)
- ▶ Configuration information (empty, CURVE=X, PERCEPTION_LUT=X:
WINDOW LEVEL=X; WINDOW CENTER=X)

3. Image Box errors

check the following:

- ▶ Image position
- ▶ Polarity (Normal, Reverse)
- ▶ Rows
- ▶ Columns
- ▶ Pixel Aspect ratio
- ▶ Bits Allocated

Agfa Perception_Luts for #2

100 = Exact Kanamori Curve (PERCEPTION_LUT= KANAMORI OR
PERCEPTION_LUT=100)

200=Exact linear Curve (PERCEPTION_LUT=LINEAR OR PERCEPTION_LUT=200)

100 to 200: Curve between Kanamori and Linear

< 100: Hyper-Kanamori curve

> 200: Hypo-Linear curve

Kodak Curve shapes for # 2

0-5