

the DiCOM box Ultrasound Quick Reference Card

To Start a New Exam:

- 1.) Press the New / End Exam key.
- 2.) Patient Name or I.D number is displayed. Press O.K. key.
- 3.) To Store Images Press the Print key on the U.S. or Press the B/W Capture A key.
- 4.) To send images to an Archive or Work Station only Press the B/W Capture B key

To End the Exam:

- 1.) Press the New/End Exam key.
- 2.) To Confirm End of Exam Press the O.K. key.

To Go Portable:

- 1.) Press the Network Status Key. The Keypad will display:
NETWORK CONNECTED Disconnect the Network?
Press the O.K. key to disconnect.
- 2.) While Portable; At Power-up the Keypad will Display:
NETWORK DISCONNECTED Connect Network?
Press the Cancel key. Follow the: to start a new exam steps.
- 3.) When the exam (s) are complete; return to the Ultrasound room and connect cables and power up the unit. The Keypad will display:
NETWORK DISCONNECTED Connect Network?
Press the O.K. key to send captured images. (All images will be sent from the portable Exam (s).

TO CHANGE THE PRINTER NETWORK DESTINATION:

- 1.) Press the Menu key.
- 2.) Select Network Destination. Press O.K. key
- 3.) Select B/W Printer or Archive Server Press O.K. key
- 4.) Select Printer or Archive Server that you want to print /store to. Press O.K key.
- 5.) Press Menu key to Return to the Main Menu (New / End Exam).